
Aspen Medical Products, Inc.
 6481 Oak Canyon, Irvine, CA 92618
 800-295-2776 • 949-681-0200
 www.aspenmp.com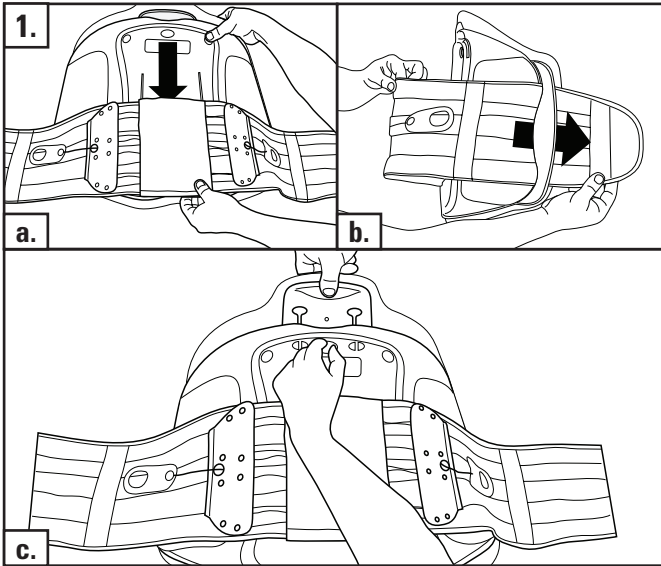

ENGLISH
ASSEMBLY

For more detailed directions about the QuikDraw RAP, please visit www.aspenmp.com.

a. Slide the large plastic tongue located on the posterior of the Contour into the pocket on the inside of the QuikDraw.

b. Slide the ends of the QuikDraw through the slots at the ends of the Contour panel. This will prevent the QuikDraw from sliding out.

c. For extra extension support, you may raise the vertical height adjustment panel in the back. Pull out the yellow rivet to disengage. When adjusted, push and snap the yellow rivet back into place.

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ENGLISH
SIZING CHART

Standard pant sizing. Size ranges are only guidelines

	MEN'S	WOMEN'S
X-SMALL	20 - 24"	child - 0
SMALL	24 - 28"	1 - 3
MEDIUM	28 - 32"	5 - 9
LARGE	32 - 36"	11 - 15
X-LARGE	36 - 42"	17 - 21
XX-LARGE	42 - 54"	23 - 33

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	MÄNNER	FRAUEN
X-SMALL	51 to 61 cm	Kind - 0
SMALL	61 to 71 cm	1 - 3
MEDIUM	71 to 81 cm	5 - 9
LARGE	81 to 91 cm	11 - 15
X-LARGE	91 to 107 cm	17 - 21
XX-LARGE	107 to 137 cm	23 - 33

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	HOMME	FEMME
X-SMALL	51 to 61 cm	enfant - 0
SMALL	61 to 71 cm	1 - 3
MEDIUM	71 to 81 cm	5 - 9
LARGE	81 to 91 cm	11 - 15
X-LARGE	91 to 107 cm	17 - 21
XX-LARGE	107 to 137 cm	23 - 33

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LIMITED WARRANTY
Aspen Medical Products, Irvine, CA 92618, warrants to the user who originally purchases this product that it is free from defects in material and workmanship. The sole obligation of Aspen Medical Products in the event of breach of warranty shall be to repair or replace the defective product or parts).
(a) The product is altered;
(b) The product is altered;
(c) Any parts not supplied by Aspen Medical Products are inserted into the product; or
(d) The product is not used in accordance with the Aspen Medical Products, Instructions for Use.

THE FOREGOING IS THE SOLE AND EXCLUSIVE REMEDY FOR FAILURE IN SERVICE OF, OR DEFECT IN, THE PRODUCT. ASPEN MEDICAL PRODUCTS SHALL NOT BE LIABLE UNDER THIS OR ANY IMPLIED WARRANTY FOR ANY DIRECT SPECIAL, INCIDENTAL, OR CONSEQUENTIAL DAMAGES. THIS WARRANTY IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED, INCLUDING THE WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE OR USE, AND ALL OBLIGATIONS OR LIABILITIES ON THE PART OF ASPEN MEDICAL PRODUCTS FOR DAMAGES ARISING OUT OF OR IN CONNECTION WITH THE USE OF THE PRODUCTS, WHICH ARE HEREBY DISCLAIMED AND EXCLUDED BY ASPEN MEDICAL PRODUCTS.
This warranty gives you specific legal rights and you may have other rights which vary from state to state.
Warning: To ensure proper use, please review all material.
A listing of authorized distributors in your area is available upon request.

READ INSTRUCTIONS BEFORE USE. Proper training in the use of this device should take place before it is applied. These directions are guidelines only and are not offered as medical recommendations.

THIS PRODUCT IS INTENDED FOR APPLICATION BY LICENSED HEALTH CARE PRACTITIONERS AS DIRECTED BY A PHYSICIAN OR OTHER QUALIFIED MEDICAL AUTHORITY. PERMANENT INJURY MAY RESULT FROM USE OTHER THAN AS DIRECTED BY A PHYSICIAN OR OTHER MEDICAL AUTHORITY. ASPEN MEDICAL PRODUCTS SPECIFICALLY DISCLAIMS ANY OBLIGATION OR LIABILITY FOR INJURY OR DAMAGE BY REASON OF ANY APPLICATION OF THIS PRODUCT. PLEASE SEE THE ACCOMPANYING LIMITED WARRANTY FOR FURTHER INFORMATION IN THIS REGARD.

PATENT PENDING
MADE IN CHINA