ENGLISH
PLACEMENT

- Apply the fully stretched brace making sure the back panel is centered and low on the back.
- Overlap the tapered ends of the brace so the hook Velcro grips the opposite side as tightly as possible.
- Ends may be angled to accommodate for different anatomies.

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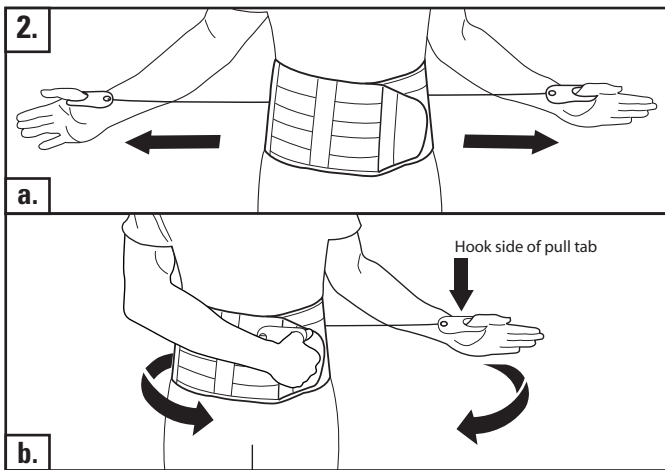
ENGLISH
TIGHTEN & ADJUST

- Slip each thumb into the FormFit™ pull tabs. Extend the tabs out and away from the brace. The left drawstring controls the compression on the lower portion (bold) of the brace, while the right drawstring controls the compression on the upper portion (bold) of the brace. Pull tabs do not have to be pulled equally, but should extend to patient's preference.
- When desired compression is achieved, touch the hook side of the pull tabs down onto the loop Velcro to secure.

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
	Extra Small	Small	Medium	Large	Extra Large	Extra Extra Large
Men's Pant Size	21 to 27 in. (46 to 61 cm)	26 to 32 in. (68 to 81 cm)	31 to 37 in. (79 to 94 cm)	36 to 42 in. (91 to 107 cm)	41 to 47 in. (104 to 119 cm)	46 to 57 in. (117 to 145 cm)
Women's Pant Size	Child - 0	2 - 4	6 - 10	12 - 16	18 - 22	24 - 34

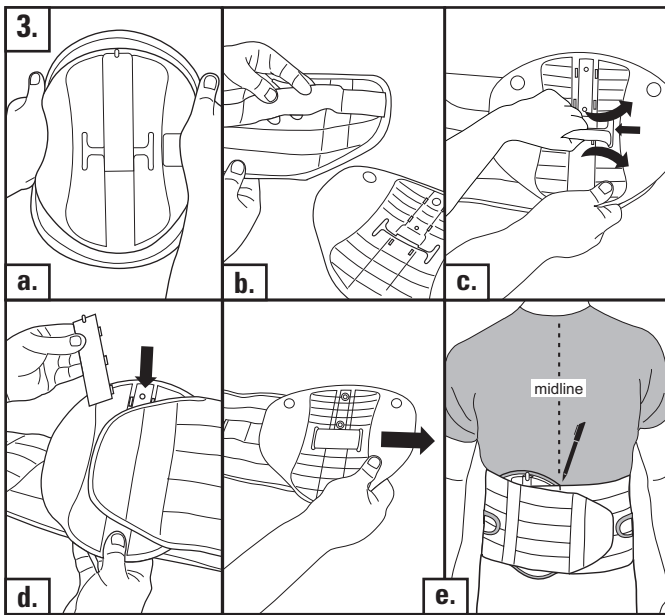*The QuikDraw PRO™ sizing follows closely to standard pants sizing. Size ranges are only guidelines. Your experiences and personal assessment could make the difference in choosing the correct size brace.

ENGLISH
RIGID ANTERIOR PANEL

- Flex the Rigid Anterior Panel (RAP) so that the center coverplate (with the midline indicator) snaps off. Remove coverplate and foam pad.
- Locate the 1.5" loop strap on the inside of the left side of the QuickDraw PRO™.
- Hold the RAP as shown and feed the edge of the loop strap through the H-shape slot in the center of the RAP. Pull the strap through the slot.
- Replace the plastic coverplate into the center of the panel.
- RAP panel should be slid to midline on patient

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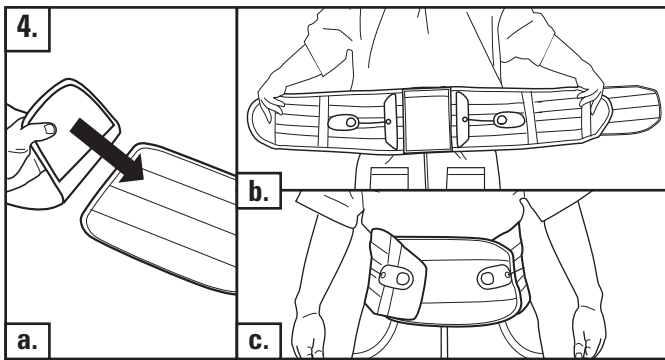
ENGLISH
EXTENSION PANEL

- Position the hook Velcro end of the QuickDraw PRO™ so the entire hook Velcro portion grips the square end of the extension panel.
- Position the brace so the pad in the back panel is centered over your spine.
- Fit and secure as instructed in section 2 (Tighten & Adjust).

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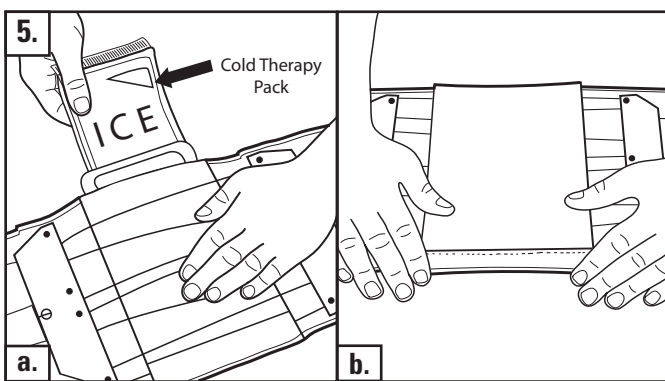
ENGLISH
COLD THERAPY

- OPTIONAL - Extra cost item
- Insert cold pack into back panel pocket. The cold pack should be placed behind the customizable rigid panel and foam pad.
 - Fold over and close the back

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LIMITED WARRANTY
Aspen Medical Products, Irvine, CA 92618, warrants to the user who originally purchases this product that it is free from defects in material and workmanship. The sole obligation of Aspen Medical Products in the event of breach of warranty shall be to repair or replace the defective product or part(s).
Aspen Medical Products shall have no obligation under this limited warranty in the event:
(a) The product was not purchased from Aspen Medical Products or through its authorized channels of distribution;
(b) The product is altered;
(c) Any parts not supplied by Aspen Medical Products are inserted into the product; or
(d) The product is not used in accordance with the Aspen Medical Products, Instructions for Use.

THE FOREGOING IS THE SOLE AND EXCLUSIVE REMEDY FOR FAILURE IN SERVICE OF, OR DEFECT IN, THE PRODUCT. ASPEN MEDICAL PRODUCTS SHALL NOT BE LIABLE UNDER THIS OR ANY IMPLIED WARRANTY FOR ANY DIRECT SPECIAL, INCIDENTAL, OR CONSEQUENTIAL DAMAGES. THIS WARRANTY IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED, INCLUDING THE WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE OR USE, AND ALL OBLIGATIONS OR LIABILITIES ON THE PART OF ASPEN MEDICAL PRODUCTS FOR DAMAGES ARISING OUT OF OR IN CONNECTION WITH THE USE OF THE PRODUCTS, WHICH ARE HEREBY DISCLAIMED AND EXCLUDED BY ASPEN MEDICAL PRODUCTS.
This warranty gives you specific legal rights and you may have other rights which vary from state to state.
Warning: To ensure proper use, please review all material.
A listing of authorized distributors in your area is available upon request.

READ INSTRUCTIONS BEFORE USE. Proper training in the use of this device should take place before it is applied. These directions are guidelines only and are not offered as medical recommendations.

THIS PRODUCT IS INTENDED FOR APPLICATION BY LICENSED HEALTH CARE PRACTITIONERS AS DIRECTED BY A PHYSICIAN OR OTHER QUALIFIED MEDICAL AUTHORITY. PERMANENT INJURY MAY RESULT FROM USE OTHER THAN AS DIRECTED BY A PHYSICIAN OR OTHER MEDICAL AUTHORITY. ASPEN MEDICAL PRODUCTS SPECIFICALLY DISCLAIMS ANY OBLIGATION OR LIABILITY FOR INJURY OR DAMAGE BY REASON OF ANY APPLICATION OF THIS PRODUCT. PLEASE SEE THE ACCOMPANYING LIMITED WARRANTY FOR FURTHER INFORMATION IN THIS REGARD.

PATENT PENDING